Ⓜ	Instructions for Use	2
Ⓜ	Ръководство за потребителя	6
Ⓜ	Návod k použití	10
Ⓜ	Brugsanvisning	14
Ⓜ	Gebrauchsanweisung	18
Ⓜ	Οδηγίες χρήσης	22
Ⓜ	Instrucciones de uso	26
Ⓜ	Kasutusjuhised	30
Ⓜ	Käyttöohjeet	34
Ⓜ	Notice d'instructions pour l'utilisation	38
Ⓜ	Treoracha úsáide	42
Ⓜ	Upute za uporabu	46
Ⓜ	Használati útmutató	50
Ⓜ	Istruzioni per l'uso	54
Ⓜ	Lietošanas instrukcijas	58
Ⓜ	Lietošanas instrukcijas	62
Ⓜ	Istruzzjonijiet għall-użu	66
Ⓜ	Gebruiksaanwijzing	70
Ⓜ	Instrukcja użytkowania	74
Ⓜ	Instruções de utilização	78
Ⓜ	Instrucțiuni de utilizare	82
Ⓜ	Návod na použitie	86
Ⓜ	Navodila za uporabo	90
Ⓜ	Bruksanvisning	94

Intended purpose: port for the subcutaneous administration of insulin.

Device description: T-Way is a port for the subcutaneous injection of insulin. It is a single-use device and is supplied sterile, in a 10-piece box. It is individually wrapped (blister pack) and is ready to use.

Intended use/Indications for use: T-Way is suitable for facilitating the subcutaneous administration of insulin via syringes or pre-loaded pens. The device can be applied to the abdomen, the upper legs, the back of the arm, the hips and the upper buttocks [Fig. 1] and can be worn for a maximum of 72 hours. The device is intended for use at home or in a health centre.

Intended users and patients: T-Way is suitable for paediatric (at least 1 year of age) and adult diabetic patients requiring multi-injection treatment. T-Way can be applied by the patient themselves or by a helper.

Clinical benefits: T-Way improves the management of multi-injection insulin treatment, avoiding fastidious multiple jabs.

Precautions for use:

- Use T-Way in accordance with the instructions of your attending physician, in particular with regards to the doses, the types and the methods of insulin administration, and the methods and frequency of blood sugar measurement.
- Carefully follow the instructions for application, use and removal as illustrated in these instructions for use.
- Use T-Way with compatible needles (of a length of between 4 and 6 mm and a diameter of between 28 and 32G).
- Use T-Way within the expiry date shown on the label and store the device in accordance with the storage indications shown on the box. The expiry date indicated is valid for products stored in their intact packaging and protected from sunlight. In other cases, sterility, integrity and performance cannot be guaranteed, leading to the risk of contamination, inefficiency, infection or allergic reaction.
- Apply T-Way to completely dry skin at least one hour after washing. Avoid applying after the use of creams or in humid environments.
- Apply T-Way in accordance with the rotation of possible application sites.
- Please note that T-Way has a maximum residual volume of 1.1 µl (0.1 units of medication). It is unknown if said residue remains in the cannula or is released gradually into the organism. Administer long-acting insulin at least 1 hour after administration of rapid-acting insulin if provided for.
- Measure blood sugar levels within 2 hours from application of a new T-Way to ascertain whether the insulin has been administered correctly.

It is advisable when applying the T-Way device via T-Bridge for the first time to do so in the presence of a physician.

Contra-indications: Do not use the device on paediatric patients younger than 1 year of age.

Undesirable side-effect: The T-Way adhesive patch has passed trials concerning biocompatibility and the absence of skin-sensitising substances and allergens. In the case of reddening, itchiness or irritation of the skin, remove the device and apply a new device in a different site. If the symptoms persist, contact your attending physician.

Warnings:

- T-Way is designed for single use. Do not use for periods of time of more than 72 hours in order to avoid skin irritation or infection or kinking of the subcutaneous cannula, or loss of adhesive patch adhesion, as well as inaccurate insulin administration in the relative injection site.
- T-Way is designed to be compatible with specific needles. Do not use the device with needles beyond the range of compatibility.
- T-Way can be used during normal, everyday activities, including while showering, bathing or swimming. After immersion in water, check that no liquid has entered the device. If water has entered, remove T-Way, dispose of correctly and apply a new device.
- Inspect the blister package and the device before using. Do not use T-Way if the packaging appears damaged or open, or if the device does not appear to be intact, as sterility and/or performance may be compromised. Do not use the device if the needle appears curved or bent or if the cannula is protruding from end of the applicator needle.
- Use the device exclusively in accordance with the intended use, the instructions for use and the precautions for use. The manufacturer is not responsible for improper use of the device.
- Do not use perfumes, solvents or deodorants on T-Way.
- When T-Way is applied, a "pinch" sensation may be felt. If more acute pain or persistent sensation is felt, contact your physician, Theras support or your retailer for the possible removal and substitution of the device, if needed.
- Avoid applying T-Way to areas where skin is folded, in areas of particularly dense hair, on tattoos, moles, infections, scars, in the presence of lipodystrophy or stretch marks, in order to not hinder the absorption of insulin. Avoid areas in close contact with clothes or accessories and/or highly strained by intense physical activity.
- Do not re-use the device in the event that application is unsuccessful.

- After having applied T-Way and before using, check that the guide needle has been correctly contained in the cap.
- After having applied T-Way, do not re-pierce the silicone septum [Fig. 3C] with the applicator needle, as this may damage the subcutaneous cannula, hindering the correct administration of insulin.
- While in use, condensation may form inside the device; this will not compromise proper function.
- In the event of inexplicably high blood sugar levels, loosening and/or shifting of the adhesive patch, significant and repeated leaks of insulin from the septum or the adhesive patch or resistance to the pen during injection, always remove the device, dispose of correctly, and apply a new device in a different injection site to the one previously or just used.

In the presence of events that differ from those described herein, or in the event of problems after having used T-Way, the patient and/or health worker is advised to report said events to the manufacturer or retailer and/or the State authorities concerned.

Device instructions for use

T-Way can be applied either with the dedicated T-Bridge accessory or manually.

Application of T-Way with the T-Bridge accessory

1. After having identified and prepared the area of application, open the T-Bridge package and remove it from the protective pouch [Fig. 6].
 2. Remove T-Way from the blister pack [Fig. 7], holding it by the needle protector part [Fig. 2B].
 3. Insert T-Way into T-Bridge [Fig. 8], slotting it into the lodging in the applicator [Fig. 5F].
 4. Remove the protective tabs from the adhesive patch [Fig.9] and the needle protector [Fig. 2B] by pushing it to one side and setting it aside for later disposal [Fig. 10].
 5. Load T-Bridge by pulling on the blue plunger [Fig. 4D] until you hear a click [Fig. 11].
 6. Slightly pinch the skin of the area of application with the fingers and position T-Bridge loaded with T-Way [Fig. 12].
 7. Press both side buttons on T-Bridge simultaneously [Fig. 4E] to insert T-Way [Fig. 13].
 8. Press on the blue plunger on T-Bridge [Fig. 4D] to allow the patch on T-Way to adhere to the skin [Fig. 14].
 9. Pull T-Bridge away from the body and run a finger over the T-Way patch in order to ensure it is well-adhered [Fig. 15].
 10. While holding T-Way firmly in place with the fingers, manually remove the needle-holder cap [Fig. 2A] [Fig. 16] and put on the protector [Fig. 2B] before disposal [Fig. 17].
- In the event that the needle-holder cap [Fig. 2A] remains stuck inside T-Bridge, put the protector on the needle [Fig. 2B] and press the plunger [Fig. 4D] to release.

Manual application of T-Way

1. After having identified and prepared the area of application, remove T-Way from the blister pack, holding it by the needle protector part [Fig. 2B] [Figs. 18 and 19].
2. Remove the protective tabs from the adhesive patch [Fig.20] and the needle protector [Fig. 2B] by pushing it to one side and setting it aside for later disposal [Fig.21].
3. Slightly pinch the skin of the area of application with the fingers and insert T-Way by applying light manual pressure [Fig. 22].

Fix the device to the skin and remove the needle-holder cap [Fig. 2A] as described in the section "Application of T-Way with the T-Bridge accessory".

Instructions for the use and removal of T-Way

- Disinfect the septum [Fig. 3C] with a gauze and alcohol [Fig. 23].
- Penetrate the septum in the centre, slowly pushing in the needle of the pen/syringe perpendicularly; rotate the pen-syringe slightly with the fingers to facilitate entry of the needle into the injection channel, until it is fully inserted [Fig. 24]. Inject the medication and wait for 5 seconds before delicately removing the needle of the pen/syringe.
- To remove T-Way, lift the patch and delicately pull it away from the skin [Fig. 25].

Troubleshooting: If T-Way proves to be defective, exhibiting the problems described in the paragraphs above or in the case of problems not covered in the instructions for use, please call the Theras toll-free number for Italy; otherwise, contact your retailer.

Instructions for disposal: After use, T-Way must be disposed of in accordance with current regulations or guidelines/procedures in force in the hospital/clinic regarding hazardous medical waste.

Intended use: re-usable applicator for the easy application of the T-Way medical device.

Device description: T-Bridge is the re-usable applicator for T-Way. The device is distributed separately from T-Way, ready to use in a single package.

Intended use/Indications for use: T-Bridge is suitable for facilitating the application of T-Way for patients who require multi-injection treatment and who use T-Way, aiding repeatable and painless application. T-Bridge can be used multiple times for a maximum of one year from the first application. T-Bridge must be used in accordance with the instructions for use for T-Way.

Intended users and patients: As for T-Way, T-Bridge is suitable for paediatric (at least 1 year of age) and adult diabetic patients requiring multi-injection treatment who use the T-Way medical device. T-Bridge is particularly recommended for patients with needle phobia. T-Bridge can be used to apply T-Way by either the patient themselves or by a helper.

Precautions for use:

- Clean T-Bridge with a gauze and alcohol after use or in the presence of traces of blood or other contaminants.
- Store T-Bridge in a clean place and in a released state, in order to guarantee proper functioning for the intended duration.

It is advisable when applying the T-Way device via T-Bridge for the first time to do so in the presence of a physician.

Attention:

- Use the device exclusively in accordance with the intended use and under the usage conditions shown in the precautions for use. The manufacturer is not responsible for improper use of the device.
- Inspect the package and the device before using. Do not use if the packaging appears damaged or open, or if the device does not appear to be intact, as performance may be compromised.

In the presence of adverse events that differ from those described herein, or in the event of problems after having used T-Bridge, the patient and/or health worker is advised to report said events to the manufacturer or retailer and/or the State authorities concerned.

Device usage instructions: Refer to the instructions for use for application of T-Way with T-Bridge.

Troubleshooting: If T-Bridge proves to be defective, exhibiting the problems described in the paragraphs above or in the case of problems not covered in the instructions for use, please call the Theras toll-free number for Italy; otherwise, contact your retailer.

Instructions for disposal: After use, T-Bridge must be disposed of in accordance with current guidelines or procedures.

Manufacturer

Catalogue number

Batch number

Date of manufacture

Expiry date

Medical Device

Sterilized using ethylene oxide

Do not resterilize

Do not re-use

Fragile, handle with care

Keep away from sunlight

Keep dry

Consult instructions for use

Caution

Non-pyrogenic

Single patient multiple use

Single sterile barrier system

Do not use if package is damaged and consult instructions for use

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